



**FRESENIUS  
KABI**

caring for life

# Freka<sup>®</sup> Enteral Feeding Tube PRODUCT RANGE





# CONTENTS

## Gastric Feeding Tubes


- > Freka® Nasogastric Feeding Tube
- > Freka® PEG Standard Gastric
- > Freka® Gastrotube Balloon Retained Replacement Gastrostomy Tube
- > Freka® Belly Button Gastrostomy
- > Freka® Pexact II, FR 15

## Jejunal Feeding Tubes

- > Freka® Endolumina Nasojejunal Feeding Tube
- > Freka® EasyIn, ENFit
- > Freka® Trelumina, ENFit
- > Freka® Fine-Needle Catheter Jejunostomy, ENFit
- > Freka® Surgical Jejunostomy Tube
- > Freka® PEG Universal Intestinal Feeding Tube

## Accessory items

- > Freka PEG connector CH/FR 9 ENFit
- > Freka PEG connector CH/FR 15 ENFit
- > Freka PEG connector CH/FR 20 ENFit
- > Fixation plate for PEG CH/FR 9
- > Fixation plate for PEG CH/FR 15
- > Fixation plate for PEG CH/FR 20
- > Repair set for Freka PEG CH/FR 9
- > Repair set for Freka PEG CH/FR 15
- > Repair set for Freka PEG CH/FR 20
- > Freka Cap ENFit - Purple stoppers
- > Freka Y Connector for Freka Intestinal CH/FR 15
- > Freka Stoma Measuring Device
- > Freka Click adaptor for Freka PEG intestinal CH/FR 9
- > Cone adaptor
- > Freka repair set for Freka surgical jejunostomy, ENFit
- > Freka Extension set (30cm) (Right Angled)
- > Freka Extension set (30cm) (Straight)
- > EasyBag Bolus adaptor
- > Freka Extension set (100cm)
- > Freka Seldinger wire 80cm
- > Freka Tube Clamp CH/FR15 & CH/FR9
- > MCT oil ampoule 3ml
- > Nasal fixation tape

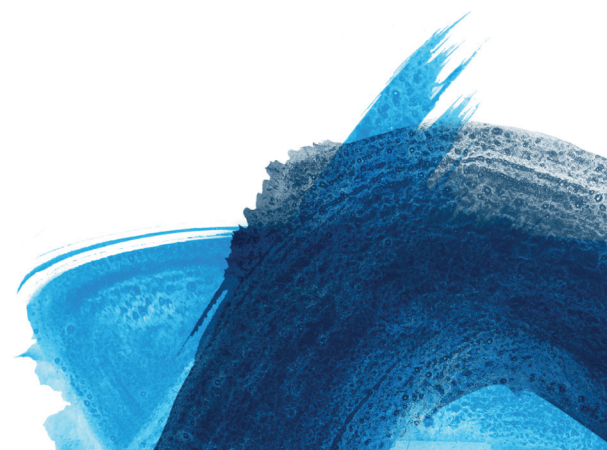


# GASTRIC FEEDING TUBES



## FREKA NASOGASTRIC FEEDING TUBE

### Nasogastric and Gastroduodenal Feeding Tube

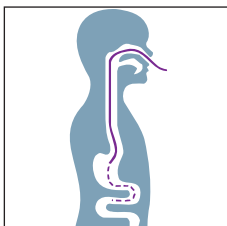


#### Key Features

- > Tissue compatible tube made from polyurethane with ENFit Connector and integral cap
- > Integrated spiral mandrin connectable to ENFit syringes
- > Flexible polyurethane olive tip with 2 openings at the distal tube end
- > 3 radio-opaque strips offering enhanced visibility
- > 1cm interval markings along length of tube
- > Three-piece nasal adhesive tape set for secure tube fixation
- > Tube marked to confirm size, manufacturer and article code

	Tube-length	External ø	Internal ø	Connector	Product code
Freka CH/FR 8 Nasogastric Feeding Tube	80cm	2.8mm	1.9mm	ENFit	7981839
Freka CH/FR 8 Nasogastric Feeding Tube	120cm	2.8mm	1.9mm	ENFit	7981840
Freka CH/FR 10 Nasogastric Feeding Tube	120cm	3.4mm	2.4mm	ENFit	7981842
Freka CH/FR 12 Nasogastric Feeding Tube	120cm	4mm	2.9mm	ENFit	7981843


Latex free  
PVC free  
DEHP free

# GASTRIC FEEDING TUBES

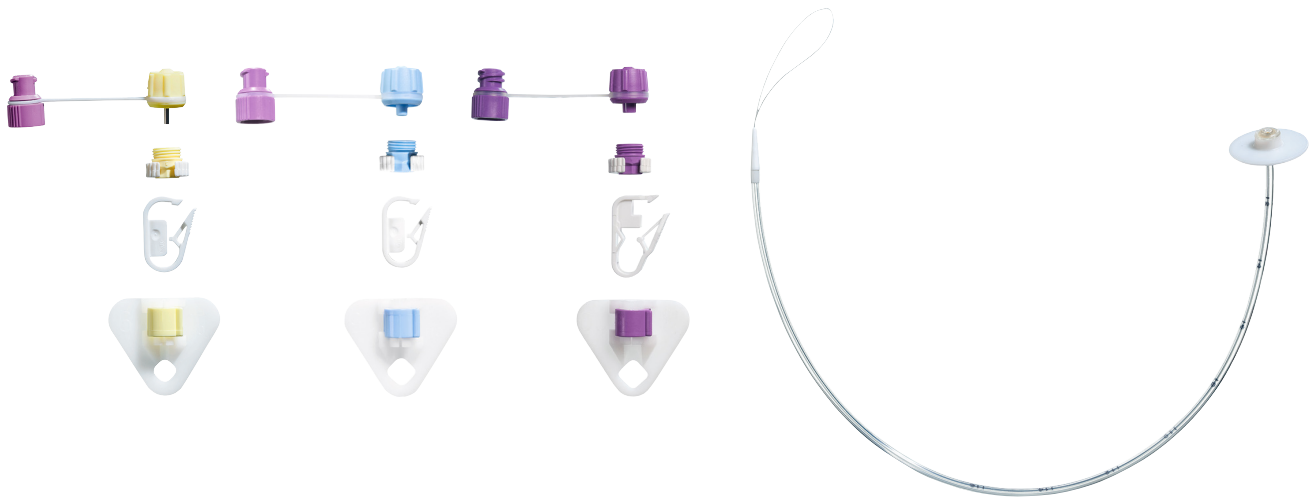

## FREKA PEG STANDARD GASTRIC

Percutaneous endoscopic gastrostomy using the pull technique

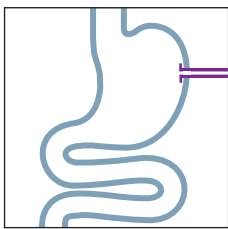

For long-term intragastric feeding

### Key Features

- > Polyurethane tube with radio-opaque strip
- > Radio-opaque retention disc made from silicone
- > Puncture cannula with safety (air) valve
- > Introducer cone with double thread
- > Safety scalpel
- > External colour and size coded fixation plate (silicone) for securing tube
- > ENFit connector
- > Interval markings every 2cm (from 2cm - 16cm)
- > Tube clamp

Latex free  
PVC free  
DEHP free


	Colour	Tube-length	External ø	Internal ø	Connector	Product code
PEG standard gastric CH/FR 9	Yellow	30cm	2.9mm	1.9mm	ENFit	7755642
PEG standard gastric CH/FR 15	Blue	35cm	4.8mm	3.6mm	ENFit	7755643
PEG standard gastric CH/FR 20	Purple	35cm	6.6mm	5.0mm	ENFit	7755644

# GASTRIC FEEDING TUBES



## FREKA GASTROTUBE

Balloon Retained Replacement Gastrostomy Tube for established stoma tract

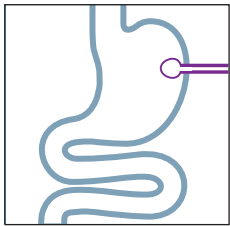

For long-term intragastric feeding


### Key Features

- > Latex free, silicone tube with retention balloon
- > External silicone retention plate
- > Separate ENFit insert to convert the funnel connector to ENFit connector after placement of feeding tube
- > Rounded distal opening
- > Long placement duration with high balloon stability
- > Insertion Introducer device
- > 10ml luer syringe (for tube inflation only)
- > Tube clamp
- > Radio-opaque strip and ring

Latex free  
PVC free  
DEHP free


	Shaft-length	External ø	Internal ø	Connector	Product code
Gastric CH/FR 15	13cm	5mm	3.6mm	ENFit	7755648

# GASTRIC FEEDING TUBES

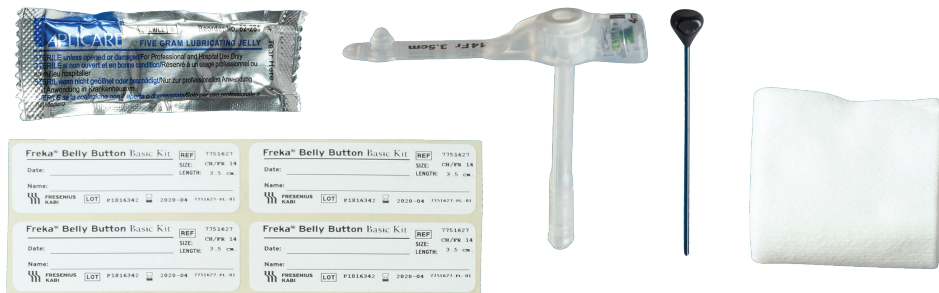

## FREKA BELLY BUTTON BALLOON GASTROSTOMY

Low profile, replacement gastrostomy for established stoma tract

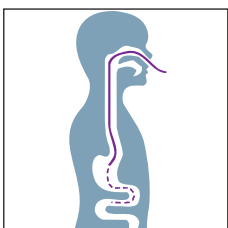

### Key Features

- > Medical grade silicone
- > Flat distal end
- > Valve for balloon filling
- > Flat and small button head
- > Anti-reflux valve
- > Introducer device for more stability during placement
- > Syringe for balloon filling


For medium to long-term intragastric feeding using a low-profile feeding device

	Shaft-length	CH/FR 12	CH/FR 14	CH/FR 16	CH/FR 18*	CH/FR 20*	CH/FR 24*
Freka Belly Button balloon gastrostomy	1.0cm	7751612	7751622	7751632	7751642	7751652	7751662
Freka Belly Button balloon gastrostomy	1.5cm	7751613	7751623	7751633	7751643	7751653	7751663
Freka Belly Button balloon gastrostomy	2.0cm	7751614	7751624	7751634	7751644	7751654	7751664
Freka Belly Button balloon gastrostomy	2.5cm	7751615	7751625	7751635	7751645	7751655	7751665
Freka Belly Button balloon gastrostomy	3.0cm	7751616	7751626	7751636	7751646	7751656	7751666
Freka Belly Button balloon gastrostomy	3.5cm	7751617	7751627	7751637	7751647	7751657	7751667
Freka Belly Button balloon gastrostomy	4.0cm	7751618	7751628	7751638	7751648	7751658	7751668
Freka Belly Button balloon gastrostomy	4.5cm	7751619	7751629	7751639	7751649	7751659	7751669
Freka Belly Button balloon gastrostomy	5.0cm	7751620	7751630	7751640	7751650	7751660	7751670

Latex free  
PVC free  
DEHP free


\*Freka Belly Button sizes CH/FR 18 CH/FR 20 CH/FR 24 may have longer delivery times.

# GASTRIC FEEDING TUBES

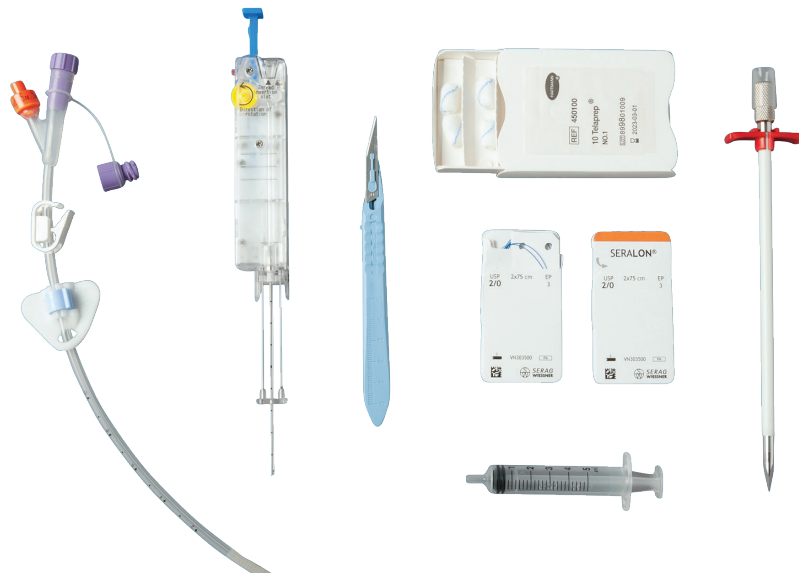

## FREKA PEXACT II, CH/FR 15

### Direct puncture gastrostomy

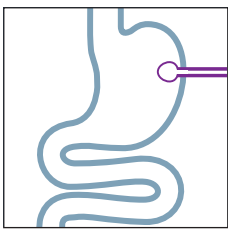

For long-term intragastric feeding

#### Key Features

- > Latex free, silicone tube, with retention balloon
- > Gastropexy device with suturing material, single handed operation
- > Improved fixation device enables single handed operation
- > Trocar with peel away sheath, CH/FR 16
- > Suturing material
- > Scalpel
- > Latex, PVC, DEHP free
- > Balloon tube with 90 day lifespan
- > Can be changed to Freka GastroTube four weeks after placement, within a fully formed and healed stoma

Latex free  
PVC free  
DEHP free

	Shaft-length	External ø	Connector	Product code
Gastric CH/FR 15	23cm +/- 1cm	5mm	ENFit	7601365




# GASTRIC FEEDING TUBES


## FREKA PEXACT II HANDLING

### Placement method Freka Pexact II with endoscopic control


1 Insertion of thread first and next Gastropexy-Device.

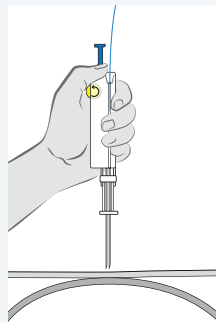


5 Trocar insertion.

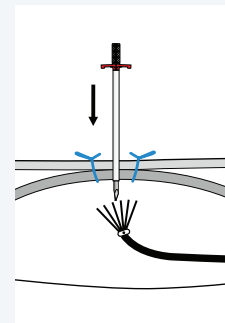

2 Push Insertion rod to open the loop-wire.

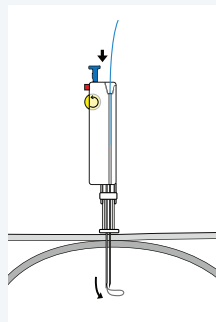


6 Tube insertion through the remaining peel-away sheath.

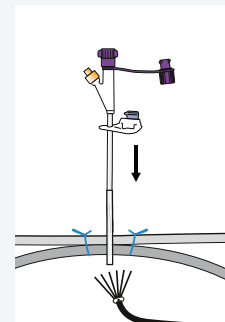


3 Pass the thread through the loop wire.


Push the release button to close the loop-wire and fix the thread.

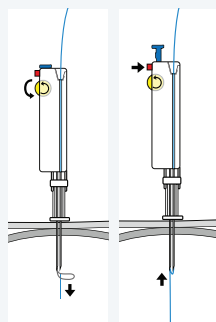


7 Block the balloon.

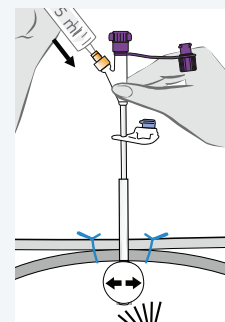


4 Remove the Gastropexy Device and make a knot.

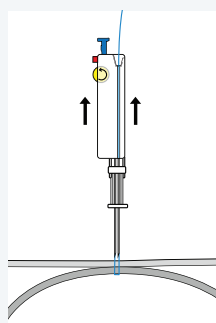

8 Peel away the sheath.

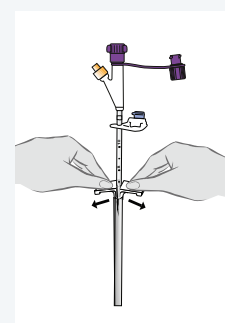


# JEJUNAL FEEDING TUBES

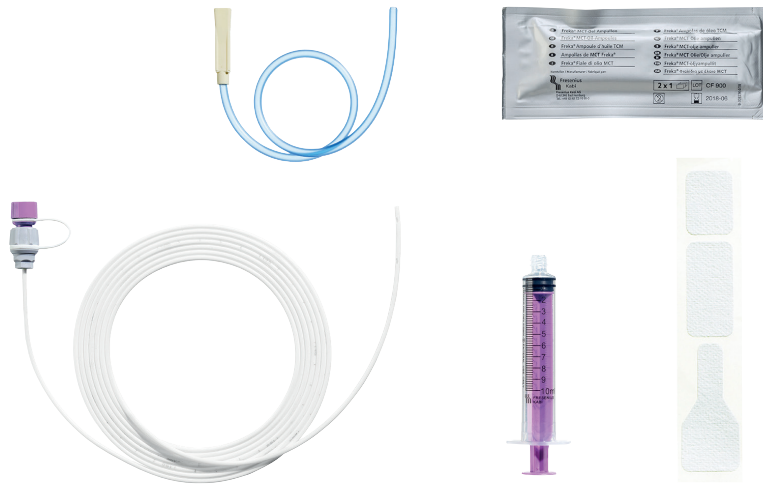

## FREKA ENDOLUMINA NASOJEJUNAL FEEDING TUBE

### Nasojejunal feeding tube

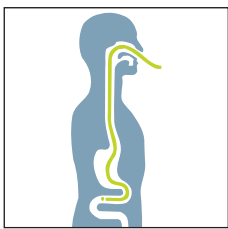

#### Single lumen feeding tube for jejunal feeding.

#### Key Features

- > Tissue compatible tube made from polyurethane for endoscopic placement via the instrumental channel of the endoscope (cut to size required after placement)
- > Four lateral side openings with round closed end
- > ENFit connector with integral cap (attached after tube placement)
- > Radio-opaque
- > Tube is numbered at 5 cm steps and marked at 1 cm steps
- > Latex, PVC, DEHP free ^
- > No guidewire needed for tube placement
- > Good X-ray visibility with barium-sulphate additive in the tube
- > Terminal rounded opening for reliable flow

#### Re-routing catheter

- > 35cm long, CH/FR 15
- > Blue

Latex free  
PVC free  
DEHP free

	Length	External ø	Internal ø	Connector	Product code
Freka endolumina	270cm	2.6mm	1.8mm	ENFit	7981832


^ Please note that whilst our Freka Endolumina, Trelumina and Easy In tubes are all DEHP free, the packs contain a re-routing catheter/tube made from PVC and DEHP which is disposed of upon tube placement.

# JEJUNAL FEEDING TUBES



## FREKA EASYIN

### Nasojejunal feeding tube

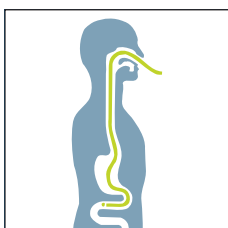

Jejunal feeding and gastric decompression

**A double lumen polyurethane tube suitable for intestinal feeding and gastric decompression/drainage**

**Key Features**

- > Tissue compatible polyurethane double lumen 270cm tube
- > Feeding lumen: 4 lateral side openings, open ended, interval markings every 1cm
- > Gastric lumen: 5 lateral side openings. Interval markings every 1cm
- > ENFit and funnel Y connector\*
- > Pack contains 2 MCT oil ampoules for lubrication
- > Blue re-routing catheter


\* ISO80369 does not specify requirements for small bore connectors when used for gastric decompression (ISO/FDIS 80369-3:2016E)

**Latex free  
PVC free  
DEHP free**

	Length	External ø	Internal ø	Connector	Product code
Freka EasyIn					7981833
Intestinal	270cm	2.6mm	1.8mm	ENFit	
Gastric	100cm	5.3mm	4.0mm	Funnel	


Please note that whilst our Freka Endolumina, Trelumina and Easy In tubes are all DEHP free, the packs contain a re-routing catheter/tube made from PVC and DEHP which is disposed of upon tube placement.

# JEJUNAL FEEDING TUBES

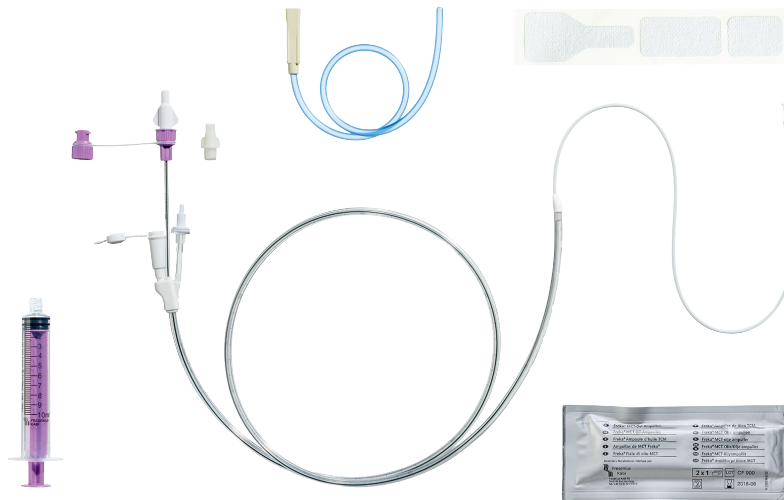

## FREKA TRELUMINA

### Nasojejunal feeding tube

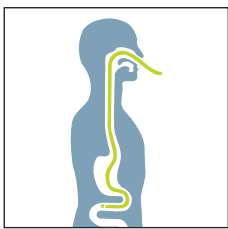

Gastric / intestinal

**A triple lumen polyurethane tube suitable for intestinal feeding and gastric decompression/drainage**

#### Key Features

- > Tissue compatible polyurethane triple lumen 150cm tube
- > Feeding lumen: Flexible olive tip with 1 distal and 2 lateral outlets ports, open ended with lumen identification marks
- > Aspiration lumen: Five large lateral outlet ports
- > Automatic pressure regulation lumen
- > Teflon coated guidewire 150cm
- > Radio-opaque strip along tube length
- > Interval markings every 1cm

	Length	External ø	Internal ø	Connector	Product code
Freka Trelumina					7981834
Intestinal feeding lumen	150cm	2.9mm	1.9mm	ENFit	
Gastric Aspiration lumen	95cm	5.3mm	4.1mm	Funnel	
Gastric Ventilation valve	95cm		1.2mm		

Latex free  
PVC free  
DEHP free


Please note that whilst our Freka Endolumina, Trelumina and Easy In tubes are all DEHP free, the packs contain a re-routing catheter/tube made from PVC and DEHP which is disposed of upon tube placement.

## JEJUNAL FEEDING TUBES



# FREKA FINE-NEEDLE CATHETER JEJUNOSTOMY

**Freka Fine-Needle Catheter Jejunostomy, ENFit - Intestinal Feeding Tube for surgical placement, for post-operative feeding.**

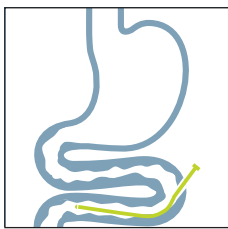

Intestinal

### Key Features

- > 10.5 cm long peel-away sheath and 13 cm peel-away sheath with blunt mandrin
- > Polyurethane tube with radio-opaque strip, open-ended ENFit connection with integral cap
- > Embedded X-ray strip
- > Transparent tube material for visual inspection of administration of nutrition and liquid
- > External fixation plate (silicone) with clamp and suture holes (for securing to the abdomen with sutures)
- > Interval markings every 5cm
- > External retention plate made of silicone rubber for high patient comfort
- > Two split cannulae (10.5cm and 13cm), blunt introducer in the longer split cannula
- > White clamp

### Freka FCJ Set FR 9, ENFit

- > For long-term intestinal feeding after abdominal or laparoscopic procedures

Latex free  
PVC free  
DEHP free


	Length	External ø	Internal ø	Connector	Product code
Freka FCJ FR 9, ENFit	75 cm	2.9 mm	1.9 mm	ENFit	7755645

## JEJUNAL FEEDING TUBES



## FREKA PEG UNIVERSAL INTESTINAL FEEDING TUBE

### Intestinal extension for the Freka PEG CH/FR15

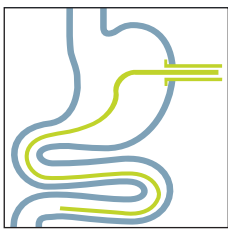

For long-term  
intra-jejunal feeding

#### Key Features


- > Intestinal extension for the gastric Freka PEG FR 15 for long term-feeding for patients at risk of aspiration or motility disorders.
- > Safe fixation with the Freka Click adapter, ENFit
- > Opening of the intestinal tube allows application by the "Seldinger Technique"
- > Polyurethane tube (120cm, CH/FR 9) with radio - opaque strip, teflon coated guidewire, olive tip and 4 lateral exit ports
- > The Freka PEG universal gastric tube CH/FR 15 acts as the introducer for the small intestine tube
- > Y connector available, for simultaneous gastric and intestinal access
- > Interval markings
  - Gastric tube every 2cm
  - Intestinal tube every 10cm

**Latex free**  
**PVC free**  
**DEHP free**

	Length	External ø	Internal ø	Connector	Product code
Freka PEG Universal Intestinal Feeding Tube					
Intestinal	120cm	2.9mm	1.9mm	ENFit	7755646



## ACCESSORIES

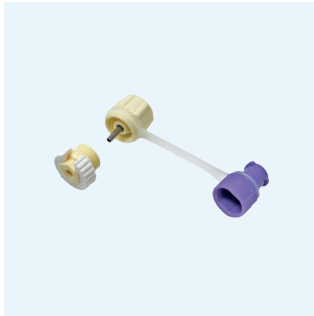


### Freka PEG Connector CH/FR 9 ENFit

> For Freka PEG Intestinal  
CH/FR 9 (yellow)



### Freka Fixation Plate

> Freka PEG CH/FR 9

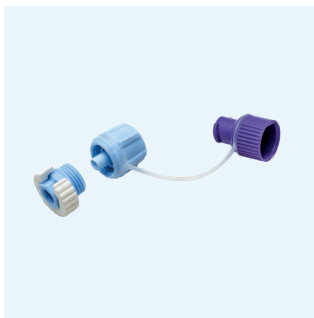


### Freka PEG Connector CH/FR 15 ENFit

> For Freka PEG CH/FR 15  
(blue)



### Freka Fixation Plate

> Freka PEG CH/FR 15

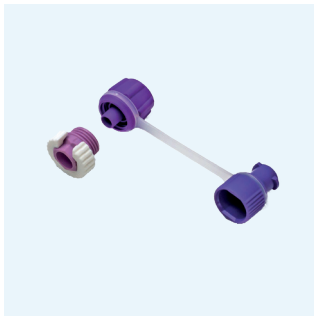

### Freka PEG Connector CH/FR 20 ENFit

> For Freka PEG CH/FR 20  
(purple)



### Freka Fixation Plate

> Freka PEG CH/FR 20

Item	Pack Size	Product Code	Item	Pack Size	Product Code
Freka PEG connector CH/FR 9 ENFit	1 x 15	7981385	Fixation plate for PEG CH/FR 9	1 x 5	7903002
Freka PEG connector CH/FR 15 ENFit	1 x 15	7981386	Fixation plate for PEG CH/FR 15	1 x 5	7904002
Freka PEG connector CH/FR 20 ENFit	1 x 15	7981387	Fixation plate for PEG CH/FR 20	1 x 5	7751541



# ACCESSORIES



### Freka Repair Set

- > For Freka PEG CH/FR 9/15/20
- > 1 external fixation plate
- > 1 clamp
- > 1 ENFit connector




### Freka Stoma Measuring Device

- > Used to measure stoma tract length prior to button insertion
- > For use when selecting Freka Belly Button  $\geq 14$  FR



### Freka CAP ENFit

- > ENFit syringe stopper

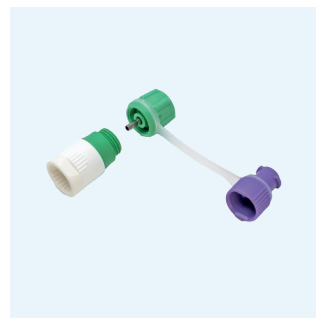


### Freka Click Adaptor For Freka PEG Intestinal

- > For Freka PEG Intestinal CH/FR 9



### Freka Y Connector for Freka PEG intestinal

- > For Freka PEG Intestinal CH/FR 15

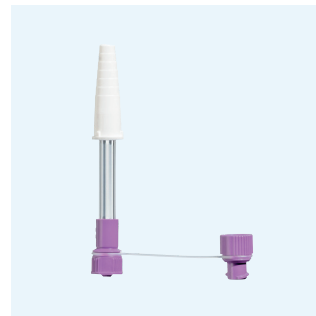

### Cone Adaptor

- > Converts funnel end into ENFit end

Item	Pack Size	Product Code
Repair set for Freka PEG CH/FR 9	1	7981382
Repair set for Freka PEG CH/FR 15	1	7981383
Repair set for Freka PEG CH/FR 20	1	7981384
Freka Cap ENFit - Purple stoppers	1 x 30	7981403
Freka Y Connector for Freka Intestinal CH/FR 15	1 x 15	7981394


Item	Pack Size	Product Code
Freka Stoma Measuring Device	1	7750921
Freka Click adaptor for Freka PEG intestinal CH/FR 9	1 x 15	7981389
Cone adaptor	1 x 15	7751562

# ACCESSORIES



**Freka repair set for Freka surgical jejunostomy, ENFit**

- > For Freka Surgical Jejunostomy CH/FR9

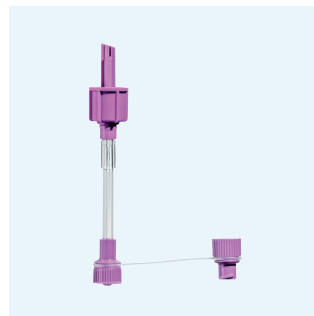

**Easybag Bolus Adaptor**

- > Resealable ENFit connection with ENPlus spiked end
- > For bolus feeding with EasyBag



**Freka extension set (30cm) (Right Angled)**

- > For use with Freka Belly Button balloon gastrostomy tube
- > Additional tube offering increased length
- > ENFit connector
- > Includes right-angle connector




**Freka extension set (100cm)**

- > For use with balloon gastrostomies
- > Additional tube offering increased length
- > ENFit connector



**Freka extension set (30cm) (straight)**

- > For use with Freka Belly Button balloon gastrostomy tube
- > Additional tube offering increased length
- > ENFit connector

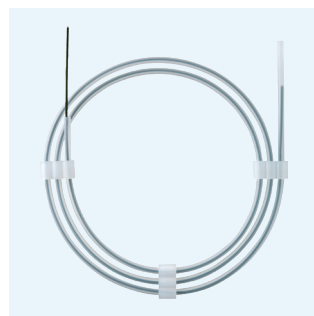

**Freka Seldinger wire 80cm**

- > Teflon-coated guidewire with flexible tip for placement of a percutaneous replacement system with the "Seldinger technique"
- > External diameter: 1.2mm

Item	Pack Size	Product Code
Freka repair set for Freka surgical jejunostomy, ENFit	1	7981388
Freka Extension set (30cm) (Right Angled)	1	7981397
Freka Extension set (30cm) (Straight)	1	7981398


Item	Pack Size	Product Code
EasyBag Bolus adaptor	1 x 15	7755694
Freka Extension set (100cm)	1	7755693
Freka Seldinger wire 80cm	1 x 10	7989911

## ACCESSORIES



### Freka Tube Clamp CH/FR15 & CH/FR9

- > Clamp for closure of the tube system.
- > Fits tube sizes FR 9 and FR 15.

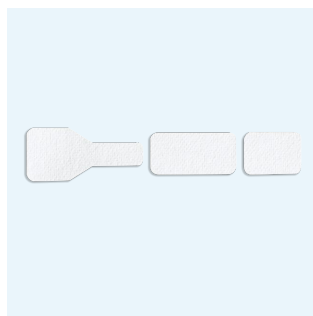

### Nasal fixation tape



### MCT oil ampoule 3ml

Item	Pack Size	Product Code
Freka Tube Clamp CH/FR15 & CH/FR9	1 x 15	7751701
MCT oil ampoule 3ml	1 x 10	3087364

Item	Pack Size	Product Code
Nasal fixation tape	1 x 50	7981907

