


**MEDICAL DEVICES**

**Hematype<sup>®</sup>**

SEGMENT SAMPLING DEVICE

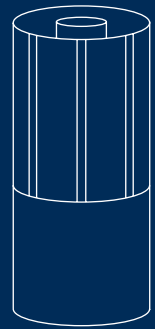

**FRESENIUS  
KABI**

caring for life

# Hematype®

Enhances SAFETY for operators

## BENEFITS & FEATURES

The Hematype® Segment Sampling Device minimizes blood splatter and eliminates the use of scissors and their potential for sharp injuries

- Eliminates the risks of using scissors and minimizes blood splatter
- Reduces sharp injuries and blood exposure to operators
- Helps you comply with the EU Directive 2000\54\EC "The protection of workers from risks related to exposure to biological agents at work"
- Additional safety is provided while minimizing operating budget impact
- Ease of use of Hematype® generates savings compared to scissor's technique
- Designed to fit both 10mm and 12mm plastic and glass test tubes

Enhancing safety for operators is making Hematype® a predominant choice for all segment sampling.

### STEP 1

Set the test tube in a rack. Place the Hematype® Segment Sampling Device on top of the test tube.



The EASE OF USE of the **Hematype® Segment Sampling Device** makes it safer for operators

### STEP 2


Cover one end of the segment with gloved hand\*, insert the other end of the segment into the Hematype® Segment Sampling Device and squeeze to obtain a sample.

(\*Avoids spray from improperly sealed segment)

### STEP 3

Recommended method of segment disposal:

Hold the Hematype® Segment Sampling Device with the segment in place and discard both into an approved biohazard medical waste container.

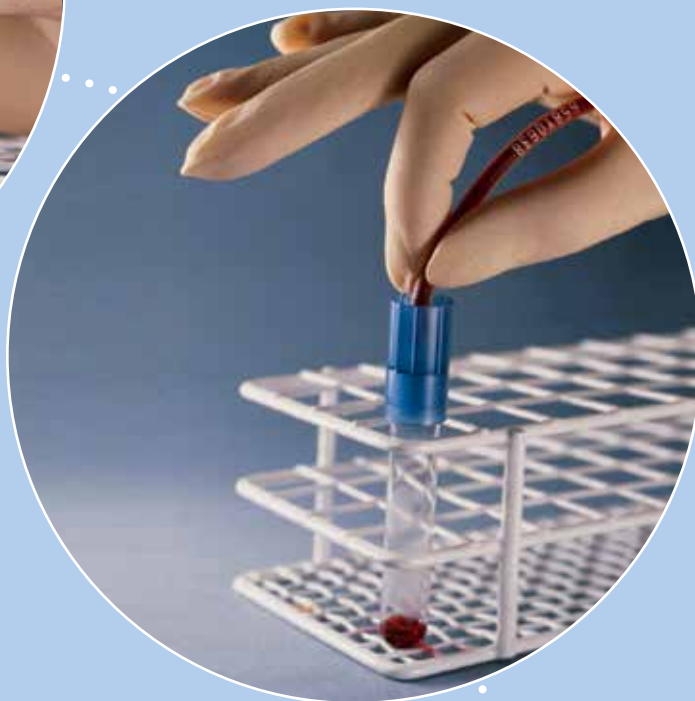

## Ordering information

For more information such as literature, technical details and working procedures as well as for equipment please contact your local sales representative or us.

### ORDER NUMBERS

---

4R5126	2000 units per carton
4R5128F	1000 units per carton
R4R5128F	1000 units per carton (CE Marked)



**Fresenius Kabi Deutschland GmbH**

Else-Kröner-Straße 1  
61352 Bad Homburg, Germany  
Phone: +49 (0) 6172 686 4721  
Fax: +49 (0) 6172 686 7259  
hematype@fresenius-kabi.com  
www.fresenius-kabi.com

The signs/names marked with ® are registered trademarks of the Fresenius Group in selected countries.

Your local sales organization: