

# How to change the Freka® PEG FR15 ENFit® Intestinal feeding tube Click Adapter.

## Changing of the Freka® Click Adapter

**1** Freka Intestinal Feeding Tube, ENFit

**2** To remove the Click Adapter squeeze the contoured surfaces of the Click Adapter firmly between two fingers

**2a** Ensure the clamp on the tube is open and pull out the intestinal tube carefully for approx. 3 - 4 cm

**2b** To prevent the intestinal tube from slipping, close the tube clamp for fixation.

**3** Cut just under the Click Adapter and remove.

## Fixation of the intestinal tube to the gastric tube

**4** Push the new Click Adapter (green/white) with its white end first over the intestinal tube.

**5** Metal pin of the Freka Connector, ENFit (green) and the **external end** of the intestinal tube must be **free of grease, clean and dry**. MCT Oil on the metal pin increases the risk of the intestinal tube detaching.

**5** Push the metal pin of the Freka Connector, ENFit **up to its base** into the intestinal tube.

**6** For fixation of the Freka Connector, ENFit and the Click Adapter **only the bottom part (green/white) should be rotated to prevent twisting of the intestinal tube.**

**6** Screw the Click Adapter (green/white) tight onto the Freka Connector, ENFit. Make sure to screw the Click Adapter until no rotation is possible.

**7** Open the tube clamp of the gastric tube and push the connected Click Adapter over the (green) intestinal access of the Freka Y-Connector, ENFit marked with an "i". The fixation must be clearly audible with a **"Click"**! Close the intestinal tube with the purple cap of the Freka Connector, ENFit.

**8**

**Warning:** A re-opened Click Adapter, cannot be used again, once the connection is detached. A proper connection cannot be generated again. A new Click Adapter must be attached.

**Warning:** When connecting the giving set to the ENFit connector it is necessary to tighten the connection by one quarter turn only. When disconnecting a giving set from the ENFit connector of the intestinal tube, grip the connector above the strap of the cap and make sure to turn the giving set connector **only** and not the tube connector.