7751612-IFU-02 - 2014-08

**FRESENIUS  
KABI**

Fresenius Kabi AG  
61346 Bad Homburg  
Germany  
Tel. +49 (0) 6172 / 686-0  
[www.fresenius-kabi.com](http://www.fresenius-kabi.com)

## ENTERAL MEDICAL DEVICES

### Freka® Belly Button Basic Kit

de - Gebrauchsinformation / en - Instruction for use / fr - Mode d'emploi / bg - Инструкция за употреба / cs - Návod k Použití / da - Brugsanvisning / el - Οδηγίες Χρήσης / es - Instrucciones de uso / fi - Käyttöohje / hr - Upute za uporabu / hu - Használati Utasítás / it - Istruzioni per l'uso / nl - Gebruiksaanwijzing / no - Bruksanvisning / pl - Instrukcja obsługi / pt - Instruções de utilização / ro - Instrucțiunile de Utilizarea / ru - Инструкция по применению / sk - Návod na použitie / sl - Upute za uporabu / sv - Bruksanvisning / tr - Uygulama kullanmak için

**Freka® Belly Button Basic Kit**

**REF**

7751612, 7751613, 7751614, 7751615, 7751616, 7751617, 7751618, 7751619, 7751620, 7751622, 7751623, 7751624, 7751625, 7751626, 7751627, 7751628, 7751629, 7751630, 7751632, 7751633, 7751634, 7751635, 7751636, 7751637, 7751638, 7751639, 7751640, 7751642, 7751643, 7751644, 7751645, 7751646, 7751647, 7751648, 7751649, 7751650, 7751652, 7751653, 7751654, 7751655, 7751656, 7751657, 7751658, 7751659, 7751660, 7751662, 7751663, 7751664, 7751665, 7751666, 7751667, 7751668, 7751669, 7751670

<b>de</b> - Freka® Belly Button Basic Kit · Perkutanes Austauschsystem	5-7
<b>en</b> - Freka® Belly Button Basic Kit · Percutaneous device	8-10
<b>fr</b> - Freka® Belly Button Basic Kit · Dispositif percutané	11-13
<b>bg</b> - Freka® Belly Button Basic Kit · Подкожна система	14-17
<b>cs</b> - Freka® Belly Button Basic Kit · Perkutánní zařízení	18-22
<b>da</b> - Freka® Belly Button Basic Kit · Perkutan enhed	21-23
<b>el</b> - Freka® Belly Button Basic Kit · Διαδερμικό εξάρτημα	24-26
<b>es</b> - Freka® Belly Button Basic Kit · Dispositivo percutáneo	27-29
<b>fi</b> - Freka® Belly Button Basic Kit · Ihonalainen laite	30-32
<b>hr</b> - Freka® Belly Button Basic Kit · Perkutani uredaj	33-35
<b>hu</b> - Freka® Belly Button Basic Kit · Perkután eszköz	36-38
<b>it</b> - Freka® Belly Button Basic Kit · Sonda di sostituzione per gastrostomia	39-41
<b>no</b> - Freka® Belly Button Basic Kit · Perkutan enhet	42-44
<b>nl</b> - Freka® Belly Button Basic Kit · Percutaan systeem	45-47
<b>pt</b> - Freka® Belly Button Basic Kit · Dispositivo percutâneo	48-50
<b>pl</b> - Freka® Belly Button Basic Kit · Urządzenie przezskórne	51-54
<b>ro</b> - Freka® Belly Button Basic Kit · Dispozitiv percutanat	55-57
<b>ru</b> - Freka® Belly Button Basic Kit · Подкожное устройство	58-60
<b>sk</b> - Freka® Belly Button Basic Kit · Perkutánná pomôcka	61-63
<b>sl</b> - Freka® Belly Button Basic Kit · Perkutani pripomoček	64-66
<b>sv</b> - Freka® Belly Button Basic Kit · Perkutan anordning	67-69
<b>tr</b> - Freka® Belly Button Basic Kit · Perkütan cihaz	70-72

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## Components

### Sterile:

- 1.1 **Freka® Belly Button (Fig. A)** with:
    - a. Feed port with anti-reflux valve (central opening)
    - b. Sealing tab
    - c. Access with valve for balloon filling (lateral opening)
    - d. Radio-opaque strip
    - e. Retention balloon
  - 1.2 **Insertion introducer**
  - 1.3 **2 compresses** (10 x 10 cm)
  - 1.4 **Syringe for filling the balloon** (1 x 1)
  - 1.5 **Lubricant gel** (1 x 1)
4. **Plastic ampoule with sterile water for filling the balloon** (1 x 10 ml)

#### Important note!

The Freka® right angle adaptor for a Luer Lock Application and the Freka® Button Extension Sets for ENLock applications as well as the Freka® Stoma length measuring device must be ordered separately (see ordering information).

### Table of contents

1. Indications
2. Contraindications
3. Determination of the stoma length and selection of the Freka® Belly Button
4. Checking the function of the Freka® Belly Button
5. Positioning the Freka® Belly Button
6. Feed administration and decompression
7. Freka® Belly Button care
8. Administration of medicines
9. Duration of use
10. How to remove the Freka® Belly Button
11. Ordering information

## 1. Indications

- Long-term intragastric feeding and medication
- Gastric decompression

For use in a mature stoma tract.

## 2. Contraindications

- Stoma not completely formed and healed.
- Length of stoma tract > 4,5 cm

#### Important notes:

The Freka® Belly Button may only be inserted as a replacement system via a well-formed and fully-healed stoma tract, no earlier than 4 weeks after initial positioning of a PEG.

In patients:

- with malnutrition or
  - being treated with steroids
- the stoma often takes longer than 4 weeks to heal completely.

## 3. Determination of the stoma length and selection of the Freka® Belly Button

#### Important notes:

- The length of the stoma tract must be determined before inserting a Freka® Belly Button in order to select a suitable Freka® Belly Button.
  - The FR size of the Freka® Belly Button should correspond to the FR size (+/- 1 FR) of the feeding tube that is already in place.
- After removing the existing feeding tube, place the stoma length measuring device following the manufacturer's

instructions for use and carefully advance into the stomach (Fig. 1).

- Measure the stoma length according to the instruction for use for the length measuring device. (Fig. 2)

The length of the stoma should be checked with the patient in supine and sitting position.

- Remove the measuring device from the stoma following the manufacturer's instructions.
- Select a Freka® Belly Button to suit the stoma length required.

Optimal is a Button length exceeding the maximal stoma length by 0.5 cm.

#### Important notes:

The Freka® Belly Button must not be shorter than the stoma length under any circumstances because of the risk of pressure necrosis!

## 4. Checking the function of the Freka® Belly Button

- Use the Luer syringe provided to draw the appropriate amount of sterile water from the plastic ampoule provided.
- Fill the balloon of the selected Freka® Belly Button with the specified amount of fluid through the valve mounted on the side (Fig.3, Table 1).
- Check the balloon visually for leaks. The filled balloon should be symmetrical. Do not press the balloon.

Freka® Belly Button FR size	Recommended filling volume
12	3.0 ml
14	4.0 ml
16	5.0 ml
18	5.0 ml
20	8.0 ml
24	8.0 ml

Table 1

- Completely withdraw the fluid from the balloon using the same Luer syringe, to make sure that the skin of the balloon lies tight to the shaft. Store the filled Luer syringe.

#### Important notes:

- The balloon may only be filled with the specified amount. Overfilling impairs the shelf life.
- After checking its function, ensure that the balloon is completely empty. The skin of the balloon lies as tight as possible to the Freka® Belly Button shaft to avoid injuries to the stoma during insertion.

## 5. Positioning the Freka® Belly Button

#### Important notes!

- In patients who have formed fistulas or in whom a long, narrow or crooked stoma is suspected, particular care and use of the Seldinger method are urgently recommended to avoid complications.
- Under no circumstances may force be used to advance the stoma length measuring device and the Freka® Belly Button. If you feel considerable resistance when advancing the tube, the Seldinger method may need to be used.

- Insert the introducer into the central opening of the Freka® Belly Button (feed port). (Fig. 4). The introducer is used to stiffen the shaft.
- Moisten the Freka® Belly Button shaft with the lubricant provided.

**Important note!**

Do not use paraffin oil or Vaseline as lubricant under any circumstances in order to avoid impairing the shelf life.

- Position the Freka® Belly Button carefully, if necessary twisting slightly, intragastrically until the head of the Freka® Belly Button lies on the skin (Fig. 5).
- Secure the Freka Button manually. Fill the balloon with the specified amount of water by connecting the Luer syringe to the lateral valve (Fig. 6). Also take note of the filling volumes from Table 1.
- Withdraw the introducer (Fig. 7) and seal the Freka® Belly Button with the sealing tab.
- Clean and dry the stoma and the surrounding skin with the compresses.

**Important notes!**

- Do not fill the balloon with air. Filling with air impairs shelf life and reduces time in position!
- Do not exceed the specified filling quantity of the balloon!
- Check by endoscopy or X-ray that the Freka® Belly Button is in the correct position!

**6. Feed administration and decompression**

Nutrition may be administered immediately after the Freka® Belly Button has been inserted.

**Important notes!**

- The Freka® right angled adapter (LuerLock) or an Freka® Button Extension Set (ENLock) should always be used to open the central feed port in order to open the anti-reflux valve. Syringes should not be attached directly to the central opening as these can damage the anti-reflux valve.
- The Freka® Button Extension Set and right angled adaptor are used following the respective manufacturer's instructions for use.

**7. Freka® Belly Button care**

- The Freka® Belly Button must be flushed with 20 ml of water before and after each feed or medicine administration or at least once a day.

**Important notes!**

- Only medication designated for feeding tube administration can be administered. Ask your doctor or pharmacist.
- Do not use fluids containing acid, particularly fruit teas or fruit juices, when feeding enterally, as these can cause the feed to coagulate and thereby block the Freka® Belly Button.
- In the event of a blockage, under no circumstances should an attempt be made to free the Freka® Belly Button using high pressure (small-volume syringe, e.g. 2 or 5 ml) or using an introducer. This poses the risk of perforation.
- Do not use paraffin oil or Vaseline.

**Stoma care**

- Always keep the outer surface of the Freka® Belly Button and the skin around the entry site dry.
- Clean daily with warm water and

mild soap. Dry thoroughly!

- The Freka® Belly Button should be rotated by 360° once a day to maintain freedom of movement within the stoma and to prevent skin irritation.
- Routinely check the entry site for redness, swelling and granulation tissue. If skin changes are noticed, consult the attending physician.

**Important notes!**

- If the stoma leaks, or if there are hardened areas or pain on pressure around the stoma during and after feed administration, it is essential for a doctor to perform an examination.
- If the Freka® Belly Button becomes noticeably loose, the fill level of the balloon should be checked.

**Checking the fill level of the balloon**

The fill level of the balloon should be checked once a week.

- Attach a Luer syringe to the lateral valve of the Freka® Belly Button and withdraw the water in the balloon completely. (Fig. 8)

Hold the Freka® Belly Button manually to prevent it slipping out of the stoma.

- Refill the balloon with the specified amount of sterile water. (Fig. 6). Also take note of the filling volumes from Table 1.
- If the filling quantity is well below the limit, the seal of the lateral valve should be checked for leakage. The volume should be rechecked within 24 hours at the latest. If the filling quantity is again below the limit, or leaks have occurred at the valve, the Freka® Belly Button should be replaced.

**Important note!**

If the Freka® Belly Button is removed inadvertently, a doctor must be consulted immediately, as the stoma can close up quickly.

**8. Administration of medicines**

Medicines should be administered in dissolved form via the Freka® right angled adapter (LuerLock) or a Freka® Button Extension Set (ENLock). Preference should be given to medication in liquid form.

**Important notes!**

- The Freka® Belly Button must be flushed with approx. 20 ml of water before and after any administration of medicines. Medicines and feed should never be mixed.
- Medicines and feed must not be administered through the valve mounted on the side for balloon filling under any circumstances.

Consult with the physician or pharmacist if the medication can be administered via a feeding tube.

**9. Duration of use**

The Freka® Belly Button can remain in situ for several months. This may vary based on different factors, e.g. the fill level of the balloon, the pH value in the stomach and care.

Frequent administration of alcoholic drinks can cause material defects of the Freka® Belly Button.

Material defects caused by medicines are rare if the Freka® Belly Button is flushed before and after each medicine administration as recommended in Items 7 and 8.

It is recommended that the Freka® Button always be replaced by specialists and medically trained personnel.

#### 10. How to remove the Freka® Belly Button

- Attach a 10 ml Luer syringe to the laterally mounted valve of the Freka® Belly Button and completely empty the balloon by aspirating the fluid (Fig. 8).
- Carefully withdraw the Freka® Belly Button from the stoma. Hold the abdomen whilst doing so (Fig. 9).
- Then apply a waterproof dressing and observe a 12-hour fast. The stoma generally closes completely in this period. It should be healed within 7 to 10 days.
- Medical supervision is recommended until the stoma is completely healed.

#### Ordering information

##### Freka® Belly Button Basic Kit

1 x 1 Set: contains: 1 Freka® Belly Button, 1 introducer, 1 Luer syringe, 1 x 10 ml ampoule of sterile water, 1 lubricant gel, 2 compresses

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