FRESENIUS KABI

Fresenius Kabi AG
61346 Bad Homburg
Germany
Tel. +49 (0) 61 72 / 6 86-0
www.fresenius-kabi.com

CE0123

STERILE EO

REF

7755646

120cm

2,9mm

1,9mm

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1. Description

Freka® Intestinal Tube FR 9 for PEG FR15, ENFit is intended for long-term intestinal feeding or long-term administration of medication.

Pack Contents Intestinal tube made of polyurethane FR 9, 120 cm (1 x 1 piece) with:

- Integrated Teflon-coated guide wire with fitted Y-Connector and blue fixation screw
- Radio-opaque strip
- Interval markings every 10 cm
- Distal pigtail bend tip
- 4 lateral outlet ports
- Straight open flexible tube with small antenna end

Accessories (1 x 1 piece)

- Freka® Y-Connector for FR 15, ENFit for simultaneous gastric and intestinal access consisting of:
 - a) fixation screw (blue-white) for attachment to gastric Freka® PEG FR 15, ENFit
 - b) Y-Connector with funnel adaptor for gastric decompression (white) and intestinal access (green) for fixation of the Click Adapter.
- Freka® Click adapter FR 9, ENFit consisting of:
 - a) Freka® Click Adapter
 - b) ENFit connector (with metal pin)

2. Indications

a. Long-term intestinal feeding for example in:

- Prolonged coma
- Gastric outflow obstruction
- Neurological dysphagia with risk of aspiration
- Post intestinal stenosis feeding

b. Gastric decompression/ drainage by gravity and simultaneous intestinal feeding in:

- Impaired gastric emptying

c. Long-term administration of medication

3. Contraindications

Absolute Contraindications

- Intestinal obstruction
- Intestinal atony
- Peritonitis
- Sepsis

Relative contraindications

- Immune suppression
- Radiation enteritis
- Pancreatitis
- Crohn's disease (risk of a fistula formation)
- Peritoneal carcinoma

4. Warnings

1. This is a single-use product. Do not reuse.
2. Please note the shelf life of the product
3. Before the insertion of the Freka Intestinal Tube FR 9 the Gastric Tube Freka® PEG FR 15 must be positioned according the directions for use. The Freka PEG Gastric FR 15 serves as a guide tube for the Freka Intestinal Tube FR 9.
4. To facilitate intestinal tube insertion, the tip of the tube

can be moistened with sterile water or Endo gel! Ensure that the metal pin of the ENFit Connector and the external end of the tube are free of grease, clean and dry. The use of lubricants is not required (increased risk of the intestinal tube becoming loose).

5. A long-term intestinal tube should be inserted through the gastric lumen into the small intestine; it should be slack and straight – without loops. Loops remaining in the stomach, increases the risk of tube dislocation!
6. During placement do not release the attachment of the guidewire or push the guidewire forwards or backwards. The wire could inadvertently be pushed through the tube outlets. If the tube becomes blocked, it needs to be changed.
7. To prevent the Freka® Click Adapter being opened inadvertently, the screwed connection cannot be undone without destroying the adapter. If the screwed connection is undone, a new Freka Click Adapter FR 9 ENFit, article no.: 7981389 must be used.
8. Under no circumstances should the lumen of the tube be flushed forcibly (small-volume ENFit syringe, e.g. 2,5 or 5 ml) or unblocked by using a wire. There is the risk to disconnect the intestinal tube or tube perforation.
9. Do not use acid containing fluids, as they may cause it to coagulate.
10. The tube must be flushed with 20 ml water before and after each administration of medication preferably using a 20-60 ml ENFit syringe. Drugs, especially antacids, must not be mixed with feed under any circumstances.
11. Parenteral infusion pumps must not be used to administer enteral nutrition.

5. Precautions

1. Duration of Use: Based on clinical experience, the tube can remain in place for many months without complications.

Frequent administration of alcohol can result in material defects in the tube, requiring it to be changed after only a short time in situ. The tube has to be flushed with water after each administration.

Material defects caused by medicines do not generally occur if the tube is flushed before and after each administration of medication.

2. Notes on Aftercare: The gastric PEG tube should be carefully moved backwards and forwards in the stoma once a day when the puncture site has healed. The tube should not be turned under any circumstances to prevent the formation of loops and dislocation of the intestinal tube. Free movement of the tube in the stoma is important to avoid an embedding of the inner retention plate ("buried bumper syndrome").

Flush the feeding tube with 20 ml of lukewarm water before and after each administration of nutrition or at least once a day using a 20-60ml ENFit syringe. In the case of double-lumen use, the limb of the tube not being used for feeding

Freka® Intestinal Tube FR 9 for PEG FR 15,ENFit

en

should be flushed once a day in the same way.

If the tube becomes blocked, it needs to be changed. Under no circumstances should the lumen of the tube be flushed forcibly (small-volume ENFit syringe, e.g. 2,5 or 5 ml) or unblocked using a wire. **There is the risk of intestinal tube disconnection or tube perforation.**

If the intestinal tube becomes blocked, it must be replaced.

- 3. Notes on the Administration of Medicines:** Where possible, medicines should be administered in dissolved or liquid form to the T-piece on the giving set. Liquid medicines (drops or syrup) are preferred.

The tube must be flushed with 20 ml water before and after each administration of medication preferably using a 20-60 ml ENFit syringe. Drugs, especially antacids, must not be mixed with feed under any circumstances.

If there is any doubt, consultation of the doctor or a pharmacist is recommended

6. Adverse events

Free movement of the tube in the stoma is important to avoid an embedding of the inner retention plate ("buried bumper syndrome").

7. Directions

7.1 Insertion of the Intestinal Tube

Before insertion of the Freka Intestinal Tube FR 9 the gastric tube Freka® PEG FR 15 must be positioned according the directions for use. The PEG FR 15 serves as a guide tube for the intestinal tube FR 9.

Fixation of the ENFit Connector to the Gastric (Guide) Tube

- Close the tube clamp.
- Cut off the ENFit Connector of the gastric tube (Fig. 1).
- Push the blue and white fixation screw for the ENFit-connector over the gastric tube.
- **Push the pin of the Y-Connector as far as possible onto the tube and secure with the fixation screw (Fig. 2). Pull off the screwing aid (outer white ring) in a downward direction and remove it. (Fig. 3).**

Positioning the Intestinal Tube

- Open the blue fixation screw on the intestinal tube 3-4 turns and push the guidewire before the metal insert at the distal end of the tube to stretch the pigtail bend. (Fig. 4/1).
- Fix the guidewire in position again using the fixation screw (Fig. 4/2).
- Open the tube clamp on the gastric tube.
Insert the intestinal tube carefully through the intestinal access of the Y-Connector ('I' for intestinal, green) (Fig. 5)

Insertion of the intestinal tube via the gastric lumen into the small intestine. To ensure a lasting placement of the tube should be slack and straight –without loops. Any loops remaining in the stomach increases the risk of dislocation!

- In the case of endoscopic placement, it is recommended to grasp the distal end of the tube using the following endoscopic instruments and push the tube under visual control until it has passed beyond the ligament of Treitz (Fig. 6):
 - the foreign body forceps, 2:1 teeth,
 - the two-arm gripper or
 - the three-arm polyp gripper. The tube can also be pushed using an image sensor control with the aid of the guidewire.
- Once the required position is reached, remove the guidewire (Fig. 7).
- When withdrawing the endoscope, the tube should be kept in position by using the forceps or the gripper.

During placement do not release the fixing of the guidewire or push the guidewire forwards or backwards. The guidewire could inadvertently be pushed through the tube outlets.

Fixation of the Intestinal Tube to the Gastric (Guide) Tube

- To securely position the intestinal tube, close the tube clamp.
- Cut off the intestinal tube approximately 3 - 4 cm above its exit from the gastric guide tube (Fig. 8).
- **Push the green/white ENFit Click adapter from the white end into the intestinal tube.**
- Then push the metal pin of the ENFit connector as far as possible into the intestinal tube and screw the ENFit Click adapter tightly until the stop is reached (Fig. 9).
- **Open the tube clamp .**
- **Push the connected intestinal ENFit Click adapter over the ENFit intestinal access of the Y-Connector (Fig. 10) until an audible click is heard.**

- To facilitate the intestinal tube insertion, the tip of the tube can be moistened with sterile water or Endogel®
- **Ensure that the metal pin of the ENFit Connector and the external end of the tube are free of grease, clean and dry. If MCT oil is used, there is an increased risk that the intestinal tube may come off.**

7.2 Using the Y-Connector for Double-Lumen Use of the Tube

When used for gastric decompression and intestinal feeding

- the gastric access of the Y-Connector ('g' for gastric, white) is used for decompression and
- feed is administered via the intestinal access of the Y-Connector ('I' for intestinal, green) preferably using an enteral feeding pump.

7.3 Notes on the Feeding Regimen

If the gastric PEG has been in situ for 1 - 2 hours or if food is

administered to the small intestine, the administration of nutrition can start. If the gastric PEG is simultaneously inserted with the intestinal tube and if feed is being administered intragastrically, it is recommended that a fasting period of approximately 1–2 hours is adhered. Feed should be administered slowly in the beginning and increased gradually. A standard enteral feeding pump should be used for continuous feed administration.

Parenteral infusion pumps must not be used to administer enteral nutrition.

7.4 Notes on Removal and Changing of Intestinal Tube

To remove/change the intestinal tube, squeeze the contoured surfaces of the intestinal Click Adapter firmly between two fingers and pull off (Fig. 11). After that, carefully pull out the intestinal tube and replace it with a new one if required.

To prevent the Freka® Click Adapter ENFit being opened inadvertently, the screwed connection cannot be undone without destroying the adapter. If the screwed connection is undone, a new Click Adaptor FR 9 ENFit, article no.: 7981389 must be used.

Syringe for enteral feeding		
Freka® Connect ENFit/ProNeo		
Product name	piece	Product code
20ml	100 x 1	9000785
20ml (medium box)	30 x 1	9000785S
20ml (small box)	10 x 1	9000785XS
60ml	50 x 1	9000786
60ml (medium box)	30 x 1	9000786S
60ml (small box)	10 x 1	9000786XS
100ml	30 x 1	9000787
100ml (small box)	10 x 1	9000787XS

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