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en - Product description

Polyurethane tube for enteral feeding via the nasal passage.

Information for use

Indication

For administration of enteral feeding to prevent or treat deficiencies.

Contraindication

Congenital abnormalities of the gastrointestinal tract.

Tube placement

Figure 1

Place the patient in a supine position (at an angle of 30°–45°).

Figure 2

Determine the point at which the tube will exit the nose. Using the tube, measure the distance from the nose to the ear and from the ear to the stomach (for gastric placement, 2a). For intestinal placement (2b), add an additional 30–40 cm.

The exit point as identified above (see tube labelling in cm) can be documented/marked.

Important note: Only gastric tube placement may be performed in domestic settings. Intestinal tube placement must be performed in a clinical setting by trained medical staff.

Figure 3

To check the manoeuvrability of the guide wire, hold the wire head securely and move it by rotating and inserting/retracting it gently.

Figure 4

Check the nostrils carefully to ensure they are clear of obstructions prior to inserting the tube. Apply lubricant or water to the tip of the tube. Carefully insert the tube through the nose, rotating it gently to facilitate insertion.

Figure 5

When the tube reaches the pharynx, ask the patient to swallow repeatedly.

Figure 6a

Gastric placement: Each time the patient swallows, push the tube down towards the stomach until the position identified prior to starting the procedure is reached.

Figure 6b

Intestinal placement: Place the patient in a stable lateral position and pass the tube down into the small intestinal position.

Figure 7

In clinical settings, tube placement is checked by performing an X-ray (gastric placement as shown in 10a, intestinal placement as shown in 10b)

Figure 8

In domestic settings, gastric tube placement can be checked by aspirating gastric juices and measuring their pH value using a pH strip, or by insufflating air while simultaneously auscultating the stomach ("bubbling").

Important note:

Administration of antacids can influence measurements.

Please note:

A pH value corresponding to the typical gastric pH value is not definitive proof of correct tube placement. The pH value can be influenced by the position of the tube tip, displacement, previous medication and feed administration.

Tube placement should be checked regularly.

Figure 11

Once the position of the tube has been checked, the guide wire can be retracted and removed slowly and carefully. Hold the tube securely with your fingers while doing this to prevent it from becoming dislodged.

Figure 12

Using an ENFit syringe, inject approx. 20 ml of water into the tube. Secure the ENFit syringe to the tube by rotating it gently and then apply the fluid.

Figure 13

Pass the tube behind the patient's ear in a gentle arch and fix it to the bridge of the nose and cheek.

Figure 14

The tube must be flushed regularly to prevent any blockages. Inject 20 ml of water into the tube after each administration of feed or medication.

Administering medication:

It is the responsibility of the health care professional to assess whether a particular drug is suitable for administration via the tube.

Only start to administer feed once the position of the feeding tube has been checked. It is advisable to check the position of the feeding tube before each feed.

Recommendation

Use of transnasal tubes for enteral feeding is typically medically indicated for periods of up to 4 weeks.

Percutaneous tube placement should be considered for patients requiring longer treatment.

Placement in domestic settings.

Gastric tube placement can be performed in clean domestic settings, providing that hygiene measures are taken (clean, disinfected hands, clean work surface) and gastric tube placement is checked.

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