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**LAVASORB®****GB****Instruction for use****Lavasorb - ready for use Wound Irrigation Solution and Fluid for wetting Wound Dressings****Description**

Lavasorb is a ready-to-use solution intended for cleansing and irrigation, as well as moistening and decontamination of skin wounds. It is also intended for the moistening of wound dressings. Lavasorb is applied for lavage and rinsing during surgery. Lavasorb contains polyhexanide as a preservative, which, in an ancillary manner, acts antimicrobially. Macrogol (Polyethyleneglycol 4000) serves to optimize the moistening of the wound surface by reducing the surface tension. Polyhexanide prevents the growth of bacteria and fungi in the opened solution.

**Composition**

1 litre contains:

|                            |        |
|----------------------------|--------|
| Polyhexanide               | 0,40 g |
| Macrogolum 4000            | 0,02 g |
| Sodium chloride            | 8,60 g |
| Calcium chloride dihydrate | 0,33 g |
| Potassium chloride         | 0,30 g |

Purified water to 1litre  
Theor. Osmolarity: 309 mosmol/l

**Application**

- Wound cleaning / irrigation / decontamination by applying directly on the wound.
- Wetting of compress to leave on the wound for 10-15 minutes for dissolving of biofilms.
- An antimicrobial effect in the wound is expected only where the contact time is > 30 minutes.
- Cleaning of wound by means of moistening cotton wool wads or dressings.
- Intensive irrigation of wound dressing to make dressing changes more gentle.

**Restrictions for use**

- Lavasorb - Wound Irrigation Solution must not be used for infusion or injection.
- Lavasorb must not be used in case of hypersensitization to any of the ingredients of Lavasorb.
- If solution is not clear and/or the container is damaged.
- Sufficient experience is not available in infants and babies. The use of Lavasorb in this age group should be considered only if inevitably indicated.
- Sufficient experience with the use of Lavasorb in pregnant women is not available. Lavasorb should therefore not be used during the first four months of pregnancy, and thereafter only if the expected benefits justify the potential risks.

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# LAVASORB®

GB

## Storage and handling instructions

- Do not store above 25°C. Keep away from direct sunlight.
- Keep out of the sight and reach of children.
- Do not use Lavasorb after the expiry date stated on the label.
- The solution is sterile until the bottle is opened.
- Use aseptic technique when handling and administering sterile irrigation solutions.
- Shelf-life after opening the bottle: 6 weeks.
- The bottle must be closed immediately after use and the top of the bottle must be protected from contamination or direct contact.
- Bottles that have come in contact with the wound must be thrown away after use.
- Lavasorb can be warmed to body temperature immediately prior to use.
- Any unused product should be disposed of in accordance with local requirements.
- The container is recyclable.

## Hazards / First-aid measures

Harmful if swallowed.  
Ingestion: Rinse out mouth with water.  
If more than 20 ml is swallowed induce vomiting.  
Contact a physician.

## Additional information for physicians and medical personnel

Do not use Lavasorb:

- For infusion or injection
- In the eyes or in the inner and middle ear
- In the whole area of the central nervous system and the meninges, including the intralumbal route
- For intraperitoneal lavage
- At risk of pressure-induced absorption e.g. by use of high pressure flushing systems (e.g. high pressure irrigation of medullary space)
- In aseptic joint operations (danger of cartilage damage!)

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Fresenius Kabi AG