MEDICAL DEVICES

CompoGuard®

COMBINING SAFETY WITH EFFICIENCY IN WHOLE BLOOD DONATION







The NEW CompoGuard®

MORE donations MORE flexibility



Increased number of donations per fully charged battery

- New high capacity Lithium battery technology
- Enhanced capability of approximately
 70 donations per battery charge for mobile donations
- External charging station available
- Battery easily exchangeable

Increased data management flexibility

- Donation data now able to be downloaded via dedicated Fresenius Kabi USB stick

MORE benefits

One further step towards paperless donation

- A new software feature enables the direct donor verification via name, surname and date of birth



Mobile donation system


Robust transport case for mobile donation




Comfortable working station

- The Gooseneck allows the operation of the CompoGuard® without bending down
- The large in-line tray and the tube clamp enable comfortable loading of the blood bag
- Transport case is also mobile and comfortable table during donation



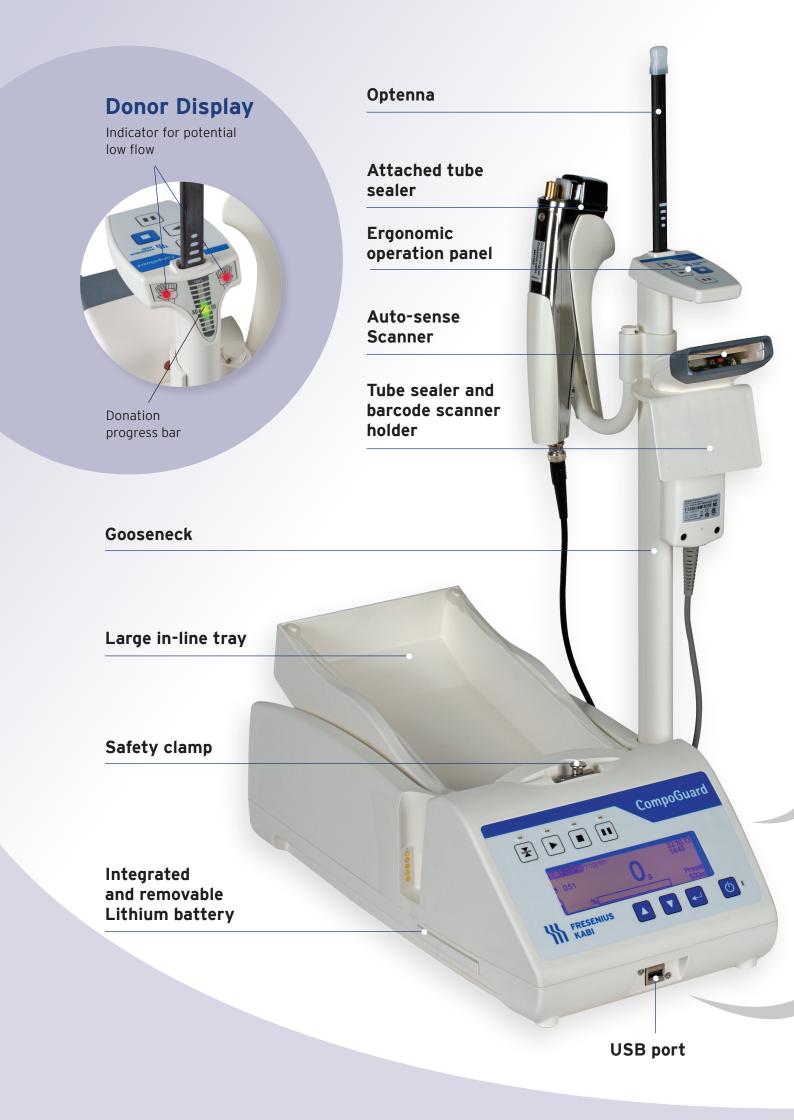

CompoMation Data Management System

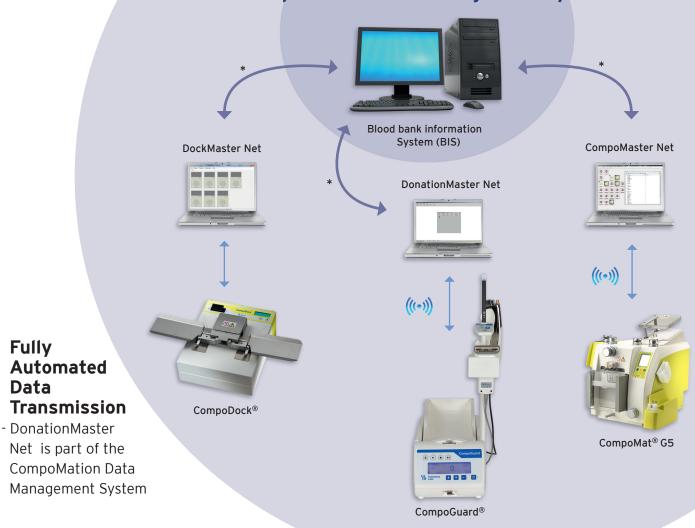



Flexible data handling with DECT² technology

* Bi-directional Interface to blood bank

- Bi-directional data transfer
- Easy and flexible programming
- Reliable data recording, surveillance and documentation



Even more flexibility with USB assistance

- In addition to DECT and cable transmission, data transfer via USB stick is now possible
- Dedicated Fresenius Kabi USB stick

² Digital Enhanced Cordless Telecommunications

Trusted features of our CompoGuard® models



Optenna (optical alarm extension) for optimal donation supervision

- The donation event can be monitored from a distance
- The blinking Optenna leads you directly to the correct donor when action is needed



Safety clamp prevents air from entering the collection bag

- The donation line tube is fixed inside the safety clamp
- The tube clamp can only be opened after successful donation and sealing
- The heat sealing is integrated and documented by the CompoGuard®



Donor engagement to improve donation process

- Donors can directly monitor their donation progress



In-line tray for optimal results

- Integrated filter holder to support proper mixing
- Large tray for optimal bag positioning and accurate weighing

Blood Flow Indication

The new CompoGuard® can be programmed to indicate high and low flow events

CompoGuard® models


2

CompoGuard® Data

- Reliable and safe bi-directional data transfer³
- Ergonomic working station

CompoGuard® Complete

- Our top model combines
- Reliable and safe bi-directional data transfer³
- Ergonomic working station
- Reliable sealing function and documentation

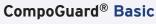

- Great choice for limited budgets







Ordering information

9029601 CompoGuard® Basic, 100-240 V

Basic model

9029611 CompoGuard® Data, 100-240 V

Basic model + DECT option + Gooseneck + USB port

9029621 CompoGuard® Complete, 100-240 V

Basic model + DECT option + Gooseneck + USB port + sealing option + rechargeable battery

³ optional cable based data management

ORDERING INFORMATION

For more information such as literature, technical details and working procedures as well as for equipment please contact your local sales representative or us.

COMPOGUARD® MODELS

9029621

9029601 CompoGuard® Basic, 100 - 240 V

Basic model (excl. battery)

9029611 CompoGuard® Data, 100 - 240 V

Basic model + DECT option + Gooseneck CompoGuard® Complete, 100 - 240 V

Basic model + DECT option + sealing option + rechargeable battery + Gooseneck

DATA MANAGEMENT ACCESSORIES

M638421 M690041	Upgrade Option DECT for model "Basic" Upgrade Option DECT for model "Basic" (USA/Canada only)
9028891	Barcode scanner with integrated auto-sense function
9028341	DECT Fixedpart for radio transmission network, EU*
	One per PC/network; includes power supply (230 V) and cable
9029001	DECT Fixedpart clone, EU*
	Fixedpart clones all have the same DECT-ID number
9029361	DECT Fixedpart for radio transmission network, USA/CDN*
	One per PC/network; includes power supply (120 V) and cable
9029461	DECT Fixedpart clone, US A/CDN*

Fixedpart clones all have the same DECT-ID number

M686791 USB adapter cable for Fixedpart

With this adapter cable the Fixedpart can be connected to a PC with USB port 9029331 Donationmaster Net 2.5.x.x multilingual

9028391

Software licence for CompoGuard® Basic

Key number for activating the data management; not necessary for models "Data" and "Complete"

9028331 Network converter RS485, EU (necessary for cable network only)* Includes power supply (230 V) and cable. One converter per PC/network 9028601 Network converter RS485, USA/CDN (necessary for cable network only)*

Includes power supply (120 V) and cable. One converter per PC/network

M636701 Y-piece (necessary for cable network only)

Number: one less than number of mixers in a network Network cable, 5 m (necessary for cable network only) M636671 Number: one less than number of mixers in a network

9029541 USB 2.0 Stick 4GB for CompoGuard®

ACCESSORIES

9029471 Transport case for CompoGuard®

Space for all accessories inside the transport case

9029481 Transport case with charging option

External plug for in-case charging of battery

Power adapter with charging indication lamp 100 - 240V, 50 - 60 Hz* Rechargeable battery for CompoGuard®, Li-Ion technology M684251

M687051

9029491 4-fold charger for CompoGuard®, EU*

For four batteries, 230 V

9029531 Gooseneck

Includes holder for scanner and hand sealer; included in models "Data" and "Complete"

M657981 Disinfectable filter holder

* additional country/language versions available upon request



caring for life

Fresenius Kabi AG

Else-Kröner-Straße 1

61352 Bad Homburg, Germany Phone: +49 (0) 61 72 686 5047 Fax: +49 (0) 61 72 686 4728 compoguard@fresenius-kabi.com www.fresenius-kabi.com

The signs/names marked with ® are registered trademarks of the Fresenius Group in selected countries.

Your local sales organization:

