



# Hematype<sup>®</sup>

## Segment sampling device

The Hematype segment sampling device allows for a safer opening of tube segments, providing enhanced safety for operators.

Reduces sharp injuries

Minimizes the risk of blood splatter and blood exposure to operators

Enhanced operator safety

# Hematype segment sampling device for enhanced operator safety

Nearly every day in hospitals and blood services, medical technical assistants may be at risk of injury from cuts and provoking cross-contaminations when the task of blood sampling is done with scissors, used to cut through the blood-filled tube segment of the blood bag for typing or cross-match.

The Hematype segment sampling device helps to eliminate the risks associated with the use of scissors for technicians and staff, providing a safer, cleaner and easier alternative.

## Better protection for a safer work environment

The Hematype segment sampling device:

- Removes the risks of splattering blood
- Eliminates the risks of cross-contamination of blood samples
- Reduces sharp injuries and blood exposure to technicians
- Helps you comply with the EU Directive 2000/54/EC  
"The protection of workers from risks related to exposure to biological agents at work"
- Fits on 10 mm and 12 mm plastic and glass test tubes

# Using the Hematype segment sampling device is as simple as 1-2-3



## STEP 1

Place the test tube in a rack and put the Hematype segment sampling device on top of the test tube.



## STEP 2

Cover the upper segment end with a gloved hand\* and insert the lower segment end into the Hematype segment sampling device. Squeeze to obtain a sample.

\*Avoids blood splatters caused by improperly sealed segment.



## STEP 3

Recommended method of segment disposal: hold the Hematype segment sampling device with the segment in place and discard both into an approved bio-hazard medical waste container.

# Hematype<sup>®</sup> Segment sampling device

---

Easy to use segment sampling device  
for better operator safety

**Process efficiency to  
help you achieve more**

## Ordering information

For more information such as literature, technical details, working instructions and equipment, please contact your local sales representative.

CE marked, Class I according EC-Directive 93/42/EEC, as amended by 2007/47/EC.

The signs/names marked with ® are registered trademarks of the Fresenius Group in selected countries.



**FRESENIUS  
KABI**

caring for life

Fresenius Kabi AG  
61346 Bad Homburg  
Germany

Phone: +49 61 72 608-0  
Fax: +49 6172 608-5786

hematype@fresenius-kabi.com  
www.fresenius-kabi.com

