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Steril und endotoxinfrei
SOLUTION FOR IRRIGATION

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B209003-09

B209003-09

<p>An MPS Company</p>	Customer	Fresenius Louviers France		Colours Used	
	Item Code	B209003-09		Black	
	Description	NOTICE VSL EAU KABI		Keyline (Non-Printing)	
	Component Type	Leaflet			
	Dimensions	630x148 – 148x105			
	Language	French			
	Market	Pharma			
	Proof By	KJA			
	Proof No	6	Min. Point Size	8 pt	
	Date	29/07/2020			

Approval Signature _____ Date _____

Warning! We cannot accept responsibility for any errors in this proof after approval. Whilst we take extreme care at all times to ensure accuracy to our client's brief, the final responsibility must be taken by our client.
IF YOU SIGN THIS PROOF YOU ARE SIGNIFYING FULL APPROVAL OF DESIGN AND TEXT.

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VERSYLENE® FRESENIUS
STERILE WATER

For irrigation only
NOT FOR INJECTION
Irrigation solution.
Sterile and free from bacterial endotoxins.
Qualitative and quantitative composition :
Water for injection 100 mL

Intended use :
Use in pre-operative, per-operative or post-operative for :
- Wound bathing.
- Wound irrigation or dressing.
- Materials rinsing by immersion or spraying.

Direction for use :
FOR IRRIGATION ONLY
NOT FOR INJECTION
Before use, check that the solution is clear and the container and seal are intact. Unscrew the cap. The tamper-proof ring breaks. Pour the solution onto the wound / dressing or rinse the equipment.
Use immediately after opening
Single use, do not reuse (risk of microbiological contamination of the solution).
Any unused portion of the solution must be discarded.
Do not warm above 40°C.

Containers :
Polypropylene bottle of 250 mL, 500 mL or 1000 mL.

Manufacturer :
Fresenius Kabi AG
61346 Bad Homburg – Germany

Distributor :
Fresenius Kabi Limited
Cestrian Court - Eastgate Way - Manor Park
RUNCORN
CHESHIRE WA7 1NT - UNITED KINGDOM

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 An MPS Company	Customer	Fresenius Louviers France		Colours Used
	Item Code	B209003-09		■ Black
	Description	NOTICE VSL EAU KABI		■ Keyline (Non-Printing)
	Component Type	Leaflet		
	Dimensions	630x148 – 148x105		
	Language	French		
Market	Pharma			
Proof By	KJA			
Proof No	6	Min. Point Size	8 pt	
Date	29/07/2020			
Warning! We cannot accept responsibility for any errors in this proof after approval. Whilst we take extreme care at all times to ensure accuracy to our client's brief, the final responsibility must be taken by our client. IF YOU SIGN THIS PROOF YOU ARE SIGNIFYING FULL APPROVAL OF DESIGN AND TEXT.				

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Fresenius Kabi Italia S.r.l.